

WRAPPING



- 1
- Position tray adjacent to reference line and on top of blue reinforcement zones.

If using a tray liner, place underneath the instrument tray before wrapping.



- 2
- Fold first layer over tray and cover completely.



- 3
- Gather side.



- 4
- Fold side up and secure.
 - Repeat steps 3-4 on other side.



- 5
- Gather top at both sides and fold inward.



- 6
- Fold over to cover tray.





- 8
- This is what the final tray should look like.



- 7
- Secure with indicator tape and label.

For more information for clinicians and product code ordering information, please visit:

www.halyardhealth.com
www.smart-fold.com

SMART-FOLD® H650 Sterilization Wrap			
NEW! Selection Color	Product Code	Dimensions	Applications per case
	14271	40 inches wide x 47 inches long	48
	14277	40 inches wide x 55 inches long	48
	14281	28 inches wide x 46 inches long	48
	14309	48 inches wide x 61 inches long	36
	14320	22 inches wide x 45 inches long	48

1 In critical zones when comparing SMART-FOLD® H650 to the second leading manufacturer's super heavyweight sterilization wrap.
2 Study comparing SMART-FOLD® Sterilization Wrap to ONE-STEP® Sterilization Wrap, Kurt Salmon and Associates, 2001. Comparing SMART-FOLD® H650 Sterilization Wrap to ONE-STEP® H600 Sterilization Wrap.
*Registered Trademark or Trademark of Halyard Health, Inc. or its affiliates.
© 2017 HYH. All rights reserved. SS63 C172147

UNWRAPPING

STRONGER
61% more weight¹ in critical zones to reduce holes, cuts, and tears

FASTER
86% faster inspection² to reduce OR setup time

SMARTER
43% reduction in training time



- 1**
- Break closure tape.
 - Unfold top layer.



- 2**
- Grab side tabs and unfold to sides.



- 3**
- Gently lift tabs from fabric and pull towards you.



- 4**
- Collect clean used wrap for recycling.



Blue Renew*
WRAP RECYCLING
HALYARD

For more information for clinicians and product code ordering information, please visit:

www.halyardhealth.com
www.smart-fold.com

Call 1-844-HALYARD (1-844-425-9273) in the United States and Canada.